

THIS TEST COMES WITH:



90mL Urine Collection Cup



Biohazard Bag



Lab Requisition



Shipping Box
(Return Label Attached)

COLLECTION DO'S & DON'TS

- ✓ **DO** — Clean the genital area with soap and water before collecting the sample.
- ✓ **DO** — Ship your sample as soon as possible after collection. This is optimal, however, if there is a slight delay, MicroGenDX samples are stable significantly longer than culture samples.
- ✗ **DO NOT** — Use Lidocaine gel. If collecting urine via catheter, Lidocaine gel can render sample invalid or give false negative test results.
- ✗ **DO NOT** — Take antibiotics. Patient must be off all antibiotics for 2 days prior to collection. This is optimal. However, if not possible, the test can still be run.
- ✗ **DO NOT** — Use Biofilm disrupting products. These agents may contain substances, like DNase, that can destroy bacterial DNA and inhibit your results. Consider instructing patients to pause the use of Biofilm disruptors for 24-48hrs prior to sample collection.

COLLECTION BEST PRACTICES

If possible limit consumption of liquids 60 minutes prior to taking a urine sample to avoid excessive dilution. However, the test can still be run. Consult with your provider if this interferes with your medical regimen.




Scan to Watch the URINE Sample Collection Video

URINE SAMPLE COLLECTION


STEP 1

Wash hands thoroughly. Cleanse all areas around the urinary exit with soap and water. With a clean tissue, blot off the urethra without touching any part of genital area with hands.

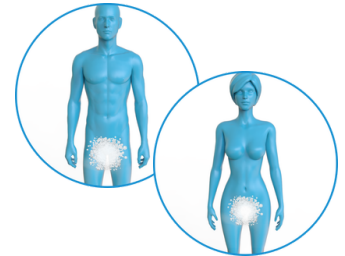

STEP 2


Spread a clean tissue on a surface close to the toilet bowl. Remove the lid from the urine cup and place it top-side down on the tissue. Place the open urine cup next to the tissue, being careful not to touch the rim of the cup or lid with fingers or body surface at any time.

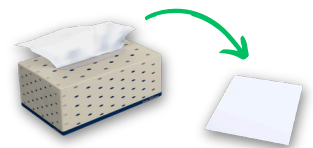

STEP 3

For Women: Before urination, separate the labia, avoid touching the urethra opening.

For Men: Before urination, retract the foreskin (if uncircumcised), avoid touching the urethra opening.

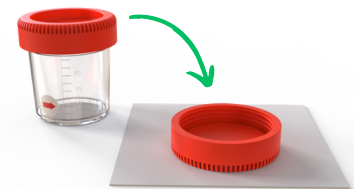

STEP 4

Hold the empty cup in one hand. Begin urination into toilet, not cup, stop urination partway through then continue urination into cup. Keep urinating into cup until the amount in the cup is level with the red arrow sticker. Do not fill the cup more than 1/2 full. Finish urinating into toilet.



STEP 5

Secure the lid tightly by placing the lid directly on the cup and turning until you hear two clicks. This indicates the cup is locked.



****If ordering UA Biomarkers please ship samples within 48 hours of collection. Sample must arrive at lab within 5 days.****

SEE PATIENT INFORMATION & SHIPPING INSTRUCTIONS ON REVERSE



Correct packaging is vital to ensuring your sample is processed in a timely manner. Please follow the steps below.

PATIENT INFORMATION

1. MARK SAMPLE

Use a permanent marker to write patient name, date of birth and sample collection date on 90mL Urine Collection Cup.



2. SIGN LAB REQ

Please have the patient sign the lab requisition form accepting financial responsibility. Be sure the physician has also signed the lab requisition form confirming consent.



2. INSURANCE INFORMATION

Submit patient face sheet and demographics with insurance information or both sides of a patient's insurance card. If this is a prepaid test skip this step.



PACKING SAMPLES FOR SHIPMENT

1. Slide the folded Lab Requisition Form into the open back pocket of the Biohazard Bag.
2. Place the 90mL Urine Collection Cup into the center sealable pocket of Biohazard Bag.
3. **IMPORTANT:** Place only one Sample and one lab requisition in each Biohazard Bag.
4. Securely seal the Biohazard Bag making sure it is completely closed.
5. Place the sealed Biohazard Bag into the MicroGenDX Shipping Box (**shipping label already attached to bottom of box*).
6. Close the MicroGenDX Shipping Box and seal with the clear sealing sticker or a piece of tape.
7. **You can add multiple completed Biohazard Bags into the MicroGenDX Shipping Box.**



Place Sample in Center Pocket



Slide Folded Lab Req into Open Pocket



3 CONVENIENT FEDEX SHIPPING METHODS

1. Drop into FedEx Dropbox including FedEx locations.
2. For Physician Offices Only: Call for pick up 1-800-GoFedEx (1-800-463-3339). Say "agent" twice to speak to agent. Let them know it's a prepaid pick up.
3. For Physician Offices Only: Use our online "Schedule a Pickup" page at www.MicroGenDX.com. **Make sure to retain your tracking number.**

